

Sol Rx 1000 Series

Full Body Panel

Dimensions:

6 feet high by 29" wide by
3 1/2" thick - 70 to 100 lbs

Electrical:

Uses standard 110-120 volt,
three prong grounded outlet.
There are no special electrical
requirements

Device Warranty:

4 years on the machine
1 year on bulbs

Arrival Warranty:

If the unit is damaged in
shipping, we will, as a
minimum, send replacement
parts at no cost




Solarc Systems is
ISO-13485 certified
for the design &
manufacturing of
medical ultraviolet
phototherapy
equipment.

US Federal law restricts this
device for sale by or on order
of a physician

All SolRx models are US-FDA
and Health Canada compliant.

SolRx™

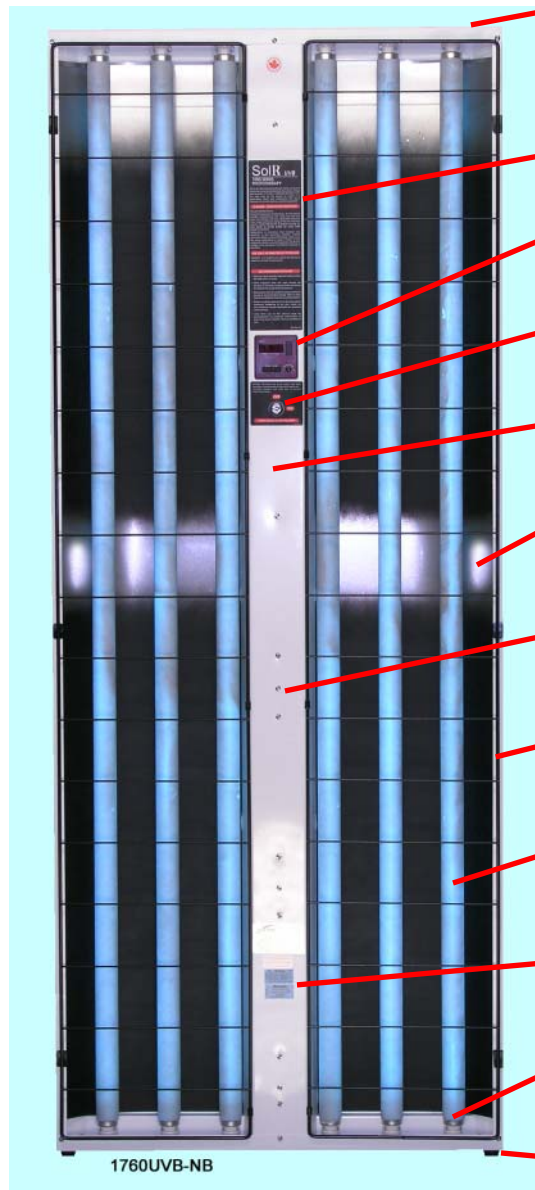

Two simple brackets mount the unit
flat on a wall or in a corner to minimize
floor space. The unit is only 3 1/2
inches thick. All hardware is supplied
Installation takes less than 10 minutes

Regulatory compliant labeling

Reliable 0-20 minute digital
countdown timer with red LED display
that is easily read with the goggles on

Keyed switchlock to prevent
unauthorized usage

Powder painted 20 gauge steel
construction securely fastened with
nyloc nuts for durability

Anodized aluminum reflectors boost
UVB light output. Parabolic sections at
outer bulbs improve peripheral output

For long term serviceability, the unit is
built with common non-proprietary
electrical components and a minimum
of plastic parts

Hinged wire guards swing out of the
way for bulb cleaning

6 foot long UVB bulbs (4,6,8,10)
Philips TL100W/01 UVB Narrowband
or FS72T12/UVB/HO UVB Broadband
These are exactly the same bulbs as
used by phototherapy clinics

3rd-Party electrical inspection labeling

Bulbs designed as low as possible to
the floor to minimize the need for a
patient platform to treat the lower leg

Rubber bumpers rest on the floor

Since 1992, thousands of SolRx 1000 Series units have been providing relief from psoriasis, vitiligo and other skin conditions. These units have become the de-facto standard for cost effective full body home UVB phototherapy.

All units are supplied fully assembled with new bulbs, UV protective goggles, hardware kit, and comprehensive User's Manual with detailed exposure guidelines for psoriasis and vitiligo. There is nothing else you need to purchase.

Four UVB Narrowband models:

1740UVB-NB: 4 bulbs, 400 watts
1760UVB-NB: 6 bulbs, 600 watts
1780UVB-NB: 8 bulbs, 800 watts
1790UVB-NB: 10 bulbs, 1000 watts

PHILIPS

Two UVB Broadband models:

1740UVB: 4 bulbs, 400 watts
1760UVB: 6 bulbs, 600 watts
Bulb types are interchangeable with labeling change

Toll Free: 1-866-813-3357
Fax: 705-739-9684
Email: info@Sol ArcSystems.com
www.Sol ArcSystems.com

12 Parker Crt, Barrie, On L4N 2A6

1000 Series Brochure Rev2.2

