

Narrowband UVB Home Phototherapy for Psoriasis, Vitiligo & Eczema

# SolRx™ 100 Series

Small Spot Treatment  
Scalp Treatment



Bulbs facing down



Bulbs facing up

Controller not shown

Model 120UVB-NB  
2 Bulbs, 18 Watts

**PHILIPS**

UVB-Narrowband


The **SolRx 100 Series** is a small UVB Narrowband device for the treatment of psoriasis, vitiligo, and eczema. It has a treatment area of 2.5 x 5 inches (12.5 square inches).

The wand is hand-held or self-supporting, and complete with the adjustable Aperture Plate™ system for accurate spot targeting. Optional accessories include the Light Brush™ for scalp treatment and Positioning Arm™ for hands-free operation.


See other side and [SolarcSystems.com](http://SolarcSystems.com) for more information.



# Sol Rx 100 Series Small spot & Scalp Treatment

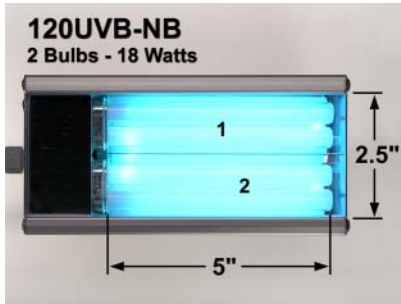

Model 120UVB-NB uses two Philips PL-S9W/01 UVB NarrowBand bulbs.

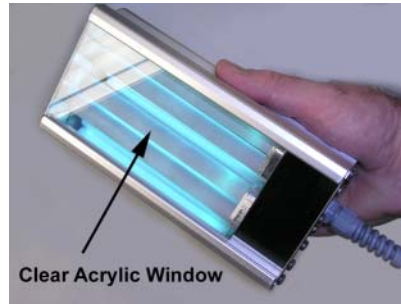


The clear acrylic window allows the wand to be placed directly on skin !



The lightweight aluminum body helps keep the device cool.



One of the many different ways to position the wand on a surface.

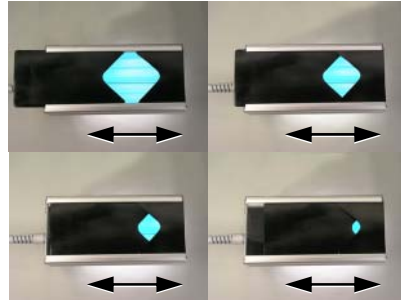

The Aperture Plate™ feature can adjust the treatment area in seconds.



The optional Light Brush™ helps clear hair so UV light can get to the scalp.



The digital timer and switchlock are important standard safety features.



The optional Positioning Arm allows hands-free use. (Desk mount shown)



The device can be operated from within its rugged carrying case.

Unlike other handheld units using only a single 9-watt bulb, the **SolRx 100 Series** can be equipped with **two 9-watt bulbs** for double the input power, double the treatment area, and a more useful treatment area shape (more square). The wand's all-metal construction and clear acrylic window allow it to be placed directly on the skin for significantly higher UVB-Narrowband light output and shorter treatment times than competitive units. It is also easier to use. All units are supplied fully assembled with new compact fluorescent bulbs, UV protective goggles, set of 6 Aperture Plates, carrying case, and comprehensive User's Manual with detailed exposure guidelines for psoriasis and vitiligo. The Light Brush™ and Positioning Arm™ are optional.

Dimensions (inches): Case 4.5 x 12 x 16    Wand: 2.25 x 3.5 x 7  
 Case weight:                    8 lbs (Model 120UVB-NB including all components)  
 Electrical:                        Uses standard 120 volt - 3 prong grounded outlet, 0.4 amps  
 Home Warranty:                 4 years on the device and 1 year on the bulbs

Two **UVB Narrowband** models:  
 120UVB-NB: 2 bulbs, 18 watts  
 110UVB-NB: 1 bulb, 9 watts  
 Also available UVB-Broadband & UVA1



All SolRx models are US-FDA and Health Canada compliant.

US Federal law restricts this device for sale by or on order of a Physician

Solarc Systems is ISO-13485 certified for the design & manufacturing of medical ultraviolet phototherapy equipment.



Toll Free: **1-866-813-3357 (705-739-8279)**  
 Fax:                                705-739-9684  
 Email :                            info@Sol ArcSystems.com  
**Sol ArcSystems.com**

1515 Snow Valley Road, Minesing, ON L0L 1Y3

100 Series Brochure Rev5.0

