




Narrowband UVB Home Phototherapy for Psoriasis, Vitiligo & Eczema

Solarx™ E-Series

EXPANDABLE !

Try one device by itself, then at any time add devices to EXPAND your system & reduce treatment times !

- Receptacles for Add-On devices
- Brackets mount the Master device flat-to-wall to minimize floor space. The device is only 3 inches thick Installation takes less than 10 minutes
- Built-in hinges connect devices together so they swing open and closed like a door (0-180 degrees)
- Reliable 0-20 minute digital countdown timer with red LED display easily seen while wearing the UV goggles supplied
- Keyed switchlock to prevent unauthorized usage
- Multi-faceted anodized aluminum reflectors boost UVB light output
- Two 6 foot long UVB bulbs per device Philips TL100W/01 UVB-Narrowband or FS72T12/UVB/HO UVB-Broadband These are exactly the same bulbs as used by phototherapy clinics
- Handles for positioning and transport Each device weighs only 33 pounds
- For long term serviceability, built with common electrical components and a minimum of plastic parts
- Powder painted 18 & 20 gauge steel construction, securely fastened with nyloc nuts for durability
- The wire guard is easily removed for access to the bulbs
- Uses standard 120V power No special electrical requirements Connect up to 5 devices per circuit
- Bulbs designed as low as possible to the floor to minimize the need for a patient platform to treat the lower leg
- Large footprint, low friction sliders glide over the floor for easy positioning of devices

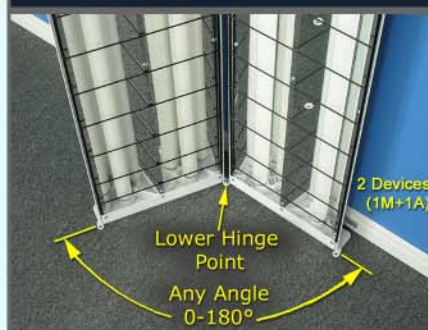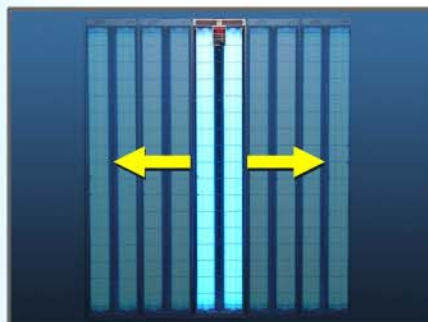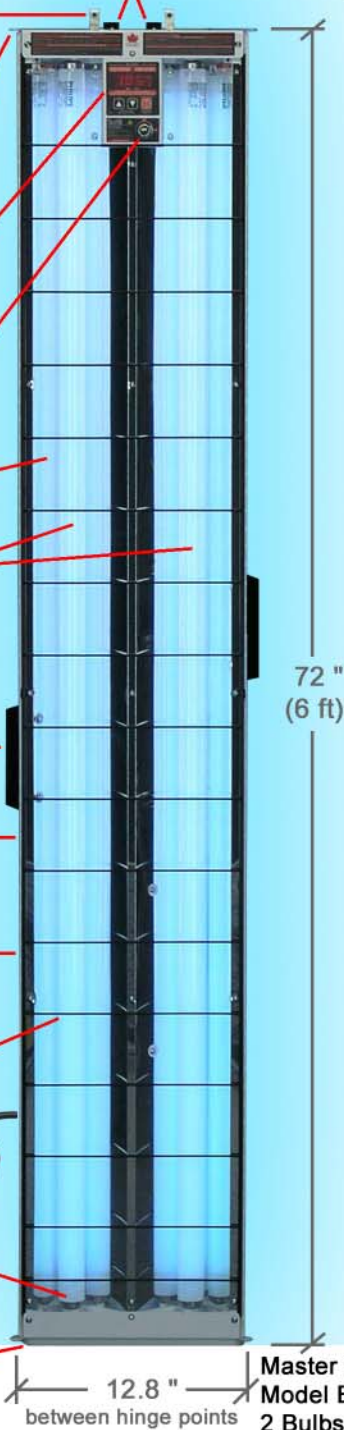

72 "
(6 ft)

12.8 "
between hinge points

Master Device
Model E720M-UVBNB
2 Bulbs, 200 Watts

PHILIPS
UVB-Narrowband

The **Solarx E-Series** is the world's only modular phototherapy system that can be expanded from a low-cost 2-bulb panel, all the way to highly efficient multidirectional wrap-around booth. It is the perfect solution for anyone that is not certain about phototherapy, or for those that simply want the best. For the treatment of psoriasis, vitiligo and eczema.


Please see SolarcSystems.com for much more information.




Sol Rx E-Series Expandable & Multidirectional



The Master device by itself is fully capable of effective UVB treatment.

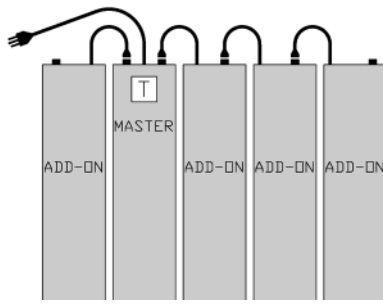

One MASTER can control up to 4 Add-On devices (5 devices total).

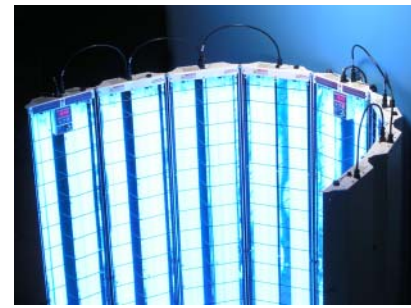

Any number of devices can be hinged together, and adjusted to any shape.



Devices fold-up nicely for storage, and quickly swing open for treatment.




The digital timer and switchlock are important standard safety features.



Several different UV phototherapy waveband bulb types can be fitted.



Some possible treatment positions; all at 8-12 inches from the bulbs.

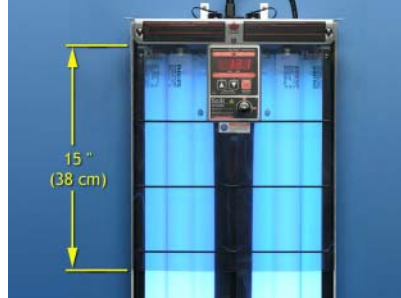

The optional Face Shield can be used to minimize UV to the face.



Handles make it easy to move. One device weighs only 33 pounds !

The SolRx **E-Series** provides countless configuration options. All devices are supplied fully assembled with new bulbs and hardware kit. Master devices include UV protective goggles, and a comprehensive User's Manual with detailed exposure guidelines for psoriasis, vitiligo and eczema. Please visit SolarcSystems.com for much more information.

Dimensions (inches): Device only: 3 x 13 x 72 high Boxed: 7 x 17 x 78
 Weight: Device only: 33 pounds (15kg) Boxed: 40 pounds (18 kg)
 Electrical: Uses standard 120Volt - 3 prong grounded outlet *
 Home Warranty: 4 years on device, 1 year on the bulbs; plus our Arrival Guarantee

All SolRx models are US-FDA and Health Canada compliant.

US Federal law restricts this device for sale by or on order of a Physician.

UVB-Narrowband Models: (All devices have two 100-watt bulbs)
 MASTER Device: E720**M**-UVBNB (complete with timer & switchlock)
 ADD-ON Device: E720**A**-UVBNB (no timer or switchlock)
 *A maximum of 4 ADD-ON devices can be connected to one MASTER.
 Each MASTER device requires its own dedicated 15-amp circuit.



Solarc Systems is ISO-13485 certified for the design & manufacturing of medical ultraviolet phototherapy equipment.



Toll Free: 1-866-813-3357 (705-739-8279)
 Fax: 705-739-9684
 Email: info@SolArcSystems.com
SolArcSystems.com

1515 Snow Valley Road, Minesing, ON L0L 1Y3

E-Series Brochure Rev3.0

